


PRACTITIONER SERIES

# Defining Roles in Your stWear™ Account Dashboard

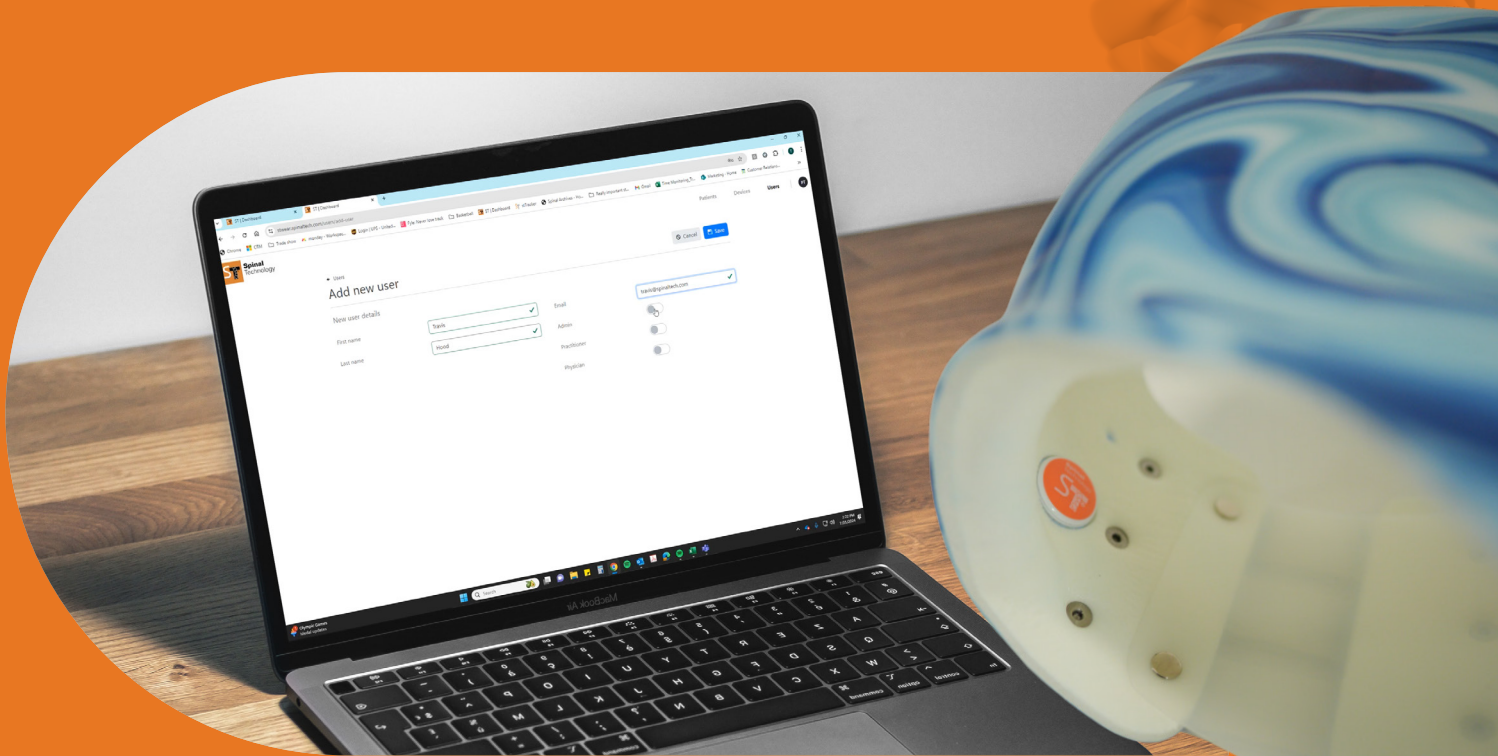

**Spinal**  
Technology

Spinal.Tech/[stWear](https://www.spinaltech.com/stWear)

This e-book is designed as a companion piece to our Online Video Training.



Visit [YouTube.com/@SpinalTechnology](https://www.youtube.com/@SpinalTechnology) to watch the full video.

## Primary Admin

Each account has one designated Primary Admin who oversees all Users\*, physicians, and patients. This User:

- Has the ability to add or remove additional Admin Users within the account.
- Provides a single, unchangeable email address for the account upon setup.
- Can add or remove Users, including Admins, Practitioners, and Physicians.
- Has the ability to add or delete patients and assign devices.
- Can log in to the stWear™ app to take readings.
- Cannot be set up as a Practitioner or Physician—this is strictly an administrative role.

\*A User is defined as another Admin, Practitioner, or Physician.

The screenshot shows a web interface for adding a new user. At the top left, there is a back arrow and the text 'Users'. The main heading is 'Add new user'. In the top right corner, there are 'Cancel' and 'Save' buttons. Below the heading, the section is titled 'New user details'. It contains four input fields: 'First name' with the value 'Travis', 'Last name' with the value 'Hood', 'Email' with a blurred address, and a role selection section. The role selection section has three toggle switches: 'Admin' (turned on), 'Practitioner' (turned off), and 'Physician' (turned off). Green checkmarks are visible in the email and role selection fields.

This is a close-up of the role selection section from the previous screenshot. It shows the 'Email' field with a blurred address and a green checkmark. Below it are three toggle switches: 'Admin' (turned on), 'Practitioner' (turned on), and 'Physician' (turned off). The 'Practitioner' toggle is also turned on in this view.



# Practitioner

With the Practitioner role\*, a User can:

- Hold both Practitioner and Admin roles (recommended for full account access).
- Add or delete only their assigned patients.
- See all patients within the account.
- Add or remove a patient’s referring Physician.

\*A User cannot be designated as a Physician if already set up as a Practitioner.

✔ User successfully added

Patients D

### Users - ST Clinic Test

+ Add user

Search in all columns...			
Name	Email	Permission level	Status
Doctor Smith	doctorsmith@spinaltech.com	Physician	Active
Jake Mele	jake_me@spinaltech.com	Practitioner Admin	Active
Jenny Netz	jenny@stb.com	Practitioner	Active
Clinician Test	physician2@spinaltech.com	Admin Practitioner	Invited
Test Physician	physician@spinaltech.com	Physician	Active
Practitioner Test	practitioner@spinaltech.com	Primary Admin	Active




## Physician

With the Physician role\*, a User can:

- Only view data for patients assigned to them.
- View patient data within the dashboard only—the mobile app is not available for Physicians.

\*This User does not have the ability to add or remove patients, assign devices, or modify account settings.

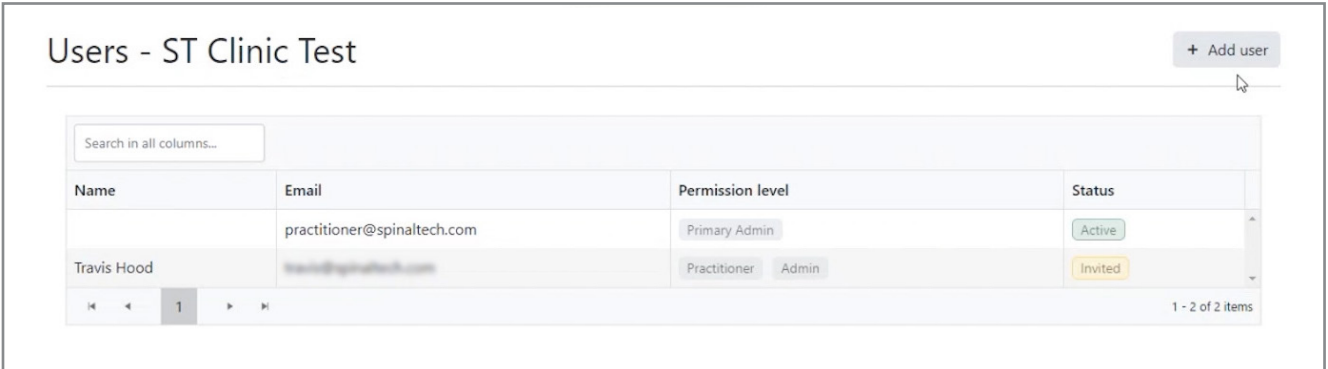

Users - ST Clinic Test

+ Add user

Search in all columns...

Name	Email	Permission level	Status
	practitioner@spinaltech.com	Primary Admin	Active
Travis Hood	travis@spinaltech.com	Practitioner Admin	Invited

1 1 - 2 of 2 items

If you need assistance, visit [Spinal.Tech/stWear](https://Spinal.Tech/stWear) to submit a support ticket or email [stWear@SpinalTech.com](mailto:stWear@SpinalTech.com).



**Spinal**  
Technology

Spinal.Tech/stWear

Phone: (508) 957-8282

