

PRACTITIONER SERIES

How to Access Patient Compliance Data



Spinal
Technology

[Spinal.Tech/stWear](https://www.spinal.tech/stWear)

This e-book is designed as a companion piece to our Online Video Training.



Visit [YouTube.com/@SpinalTechnology](https://www.youtube.com/@SpinalTechnology) to watch the full video.

Accessing Compliance Data



Any time a parent, clinician, or Admin uploads data, the “Last Upload” column will display the date and time of the recorded data. This allows easy tracking of patient compliance information.

Patients
Devices

← Patient - Training Video

Brace - Providence Brace Edit 🗑️


Brace details

Brace name	<input type="text" value="Providence Brace"/>	Target hours per day	<input type="text" value="10 hrs"/>
Assigned to patient	<input type="text" value="Training Video"/>	Target temperature range	
Current device	<input type="text" value="F989201E"/>	High Threshold	<input type="text" value="98 F"/>
Nightwear	<input checked="" type="checkbox"/>	Low Threshold	<input type="text" value="70 F"/>

Brace record

First log date	Last upload date	Total nights	Nights in compliance %
Saturday, July 20, 2024	Thursday, December 19, 2024	152	51%

1 - 1 of 1 items


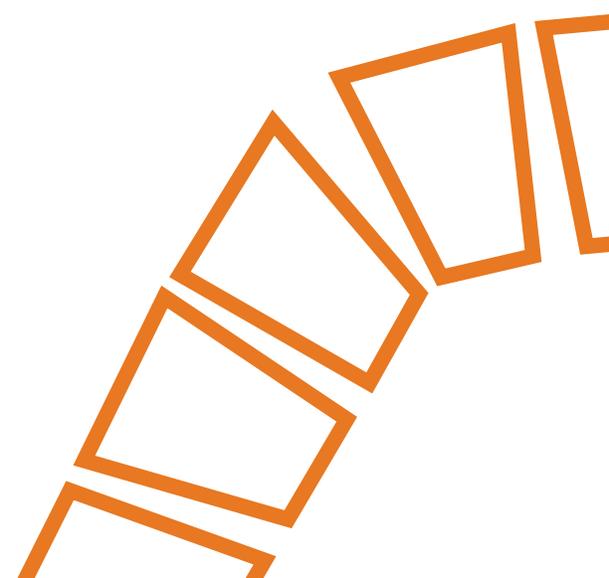


Viewing the Adherence Report

To access patient compliance details:

1. Click on the "Brace" section.
2. The adherence report will display key compliance metrics, including:

- Total nights in the brace
- Number of nights in compliance
- Longest compliance streak
- Average compliance time per night

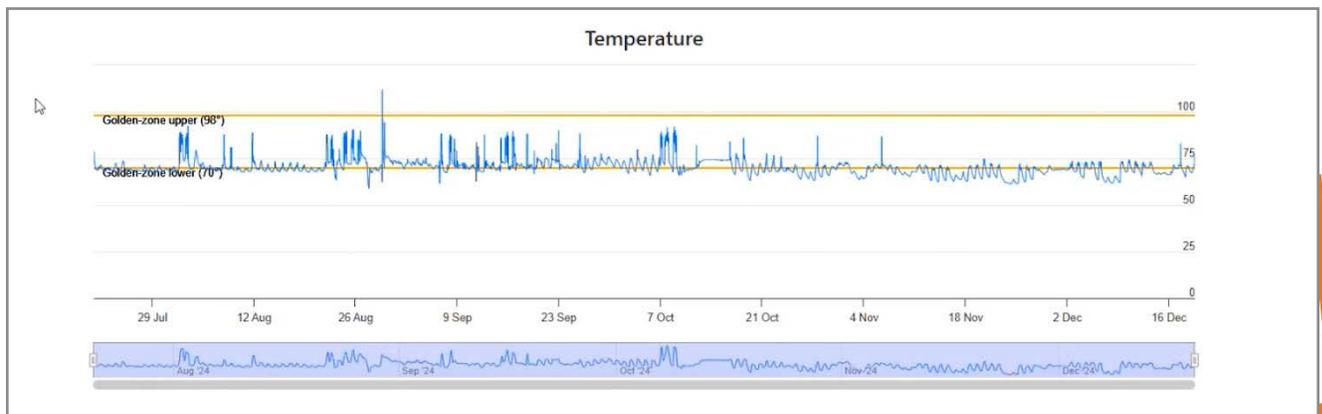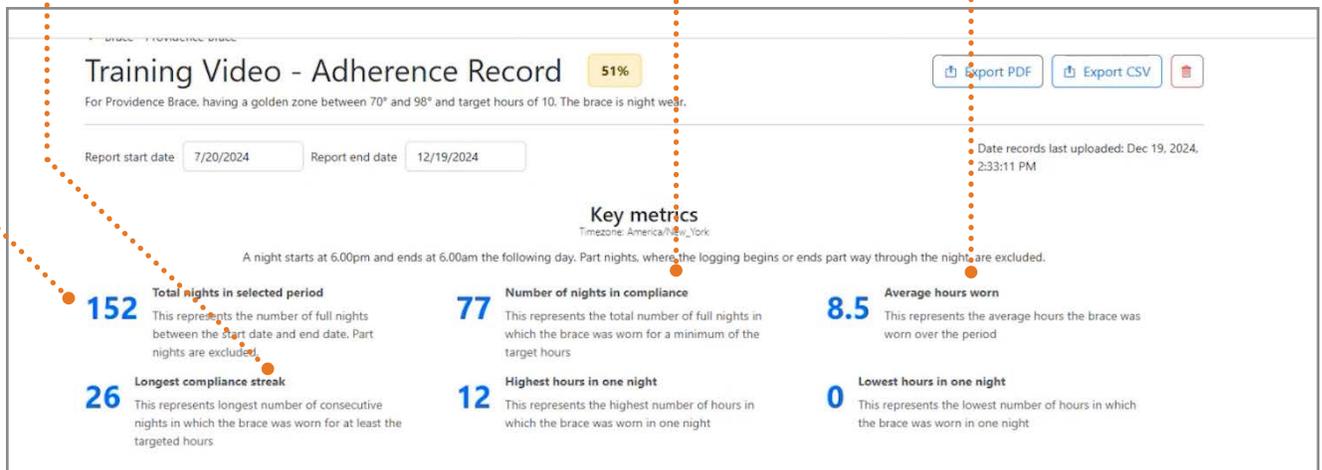

(Data displayed is for training purposes only.)

Analyzing the Data

- Clinicians can scroll through individual data points to identify patterns.
- Zoom functionality is available to troubleshoot potential temperature discrepancies.


Exporting Data

You can export the data as a PDF to be sent to the parents if needed. If an issue arises and further review is needed, you can export the data for analysis.

Steps to Export Data:

1. Click “Export CSV” to download the file.
2. Remove any identifiable information to ensure patient privacy.
3. Email the CSV file to **STWear@spinaltech.com** and explain the issue.

Our team will review the data and respond promptly with any insights or recommendations.

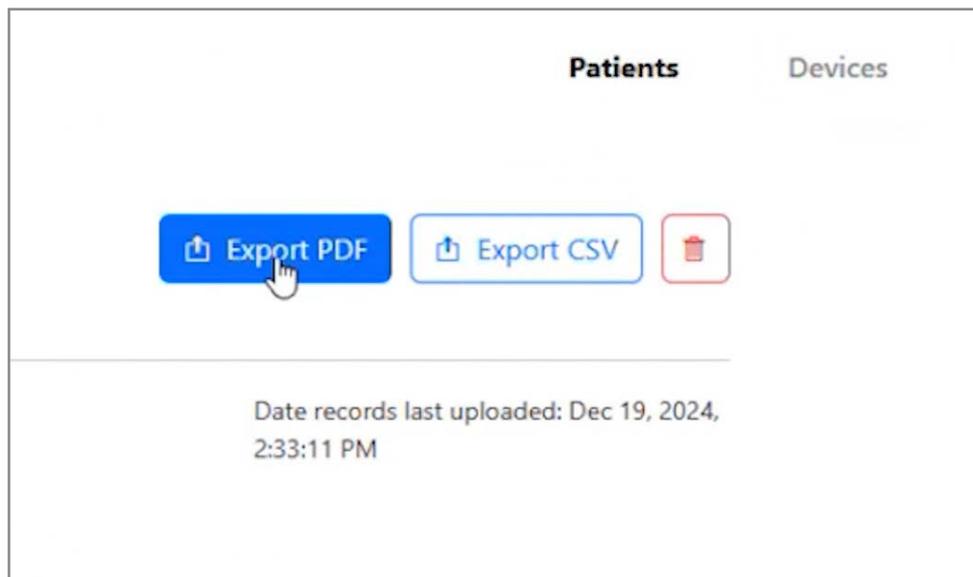



Spinal.Tech/stWear

Phone: (508) 957-8282

